1.

Depending on the direction of force required to adjust the displaced/impacted tooth, an anchorage point is selected on the space holder spring. A ligature cutter is carefully inserted here and closed slightly. This creates a space within the spring in which the ligature wire can be stored and no longer slip.

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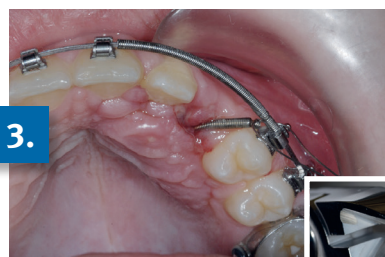
2.

The tension spring end is aligned with the anchoring position on the arch and shortened until the distance to the anchoring point is approx. 2 mm. This distance from the arch enables a force of approx. 0.3 N.

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3.

At the end of the spring, a piece approx. 1 mm (3-4 turns) wide is cut off with the ligature cutter, closed slightly and carefully bent at an angle of approx. 45° to create an loop. A ligature wire is passed through the loop.

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4.

The ligature wire is ligated at the attachment point. **After every 4 weeks**, further reactivation should be carried out with the described steps until the tooth has erupted.

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5.

When the end of the appliance with the lingual button is well broken through, it is removed from the clip with How- or Weingart forceps by carefully pulling the lingual button. If the lingual button comes loose in the process, a new lingual button must be glued to the same palatal position.

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6.

The tooth is moved further buccally with a U-shaped C-chain and derotated. A button or bracket can now be bonded buccally for further extrusion.

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
Developed by | Entwickelt von Dr. Michael Schubert, Germany


Palatinal | Palatinal

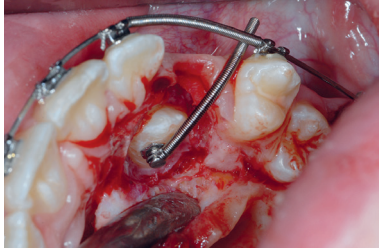In the PALATAL POSITION of the displaced tooth, after forming a mucoperiosteal flap and exposing the clinical crown, the button of the spring system is placed in the centre of the PALATINAL crown. Remove excess adhesive material! After the adhesive material has set, a trial activation should be carried out by carefully pulling on the end of the spring.

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
Buccal | Bukkal

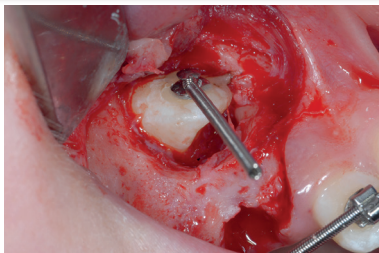In the BUCCAL POSITION of the displaced tooth, the button of the spring system is placed in the centre of the crown BUKKAL after formation of a mucoperiosteal flap and exposure of the clinical crown. Remove excess adhesive material! After the adhesive material has set, a trial activation should be carried out by carefully pulling on the end of the spring.

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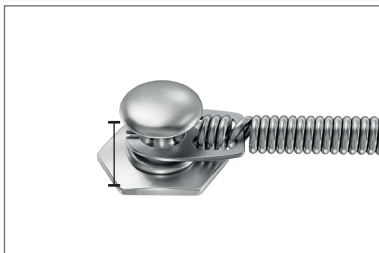
Before the adhesive hardens, any excess adhesive material between the button and the clip must be carefully removed! Free rotation is absolutely essential for the EWC system to function.

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Palatinal | Palatinal

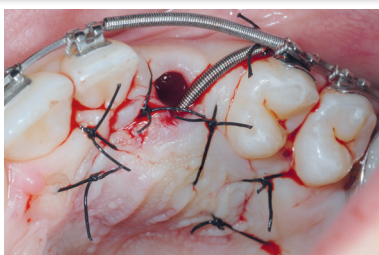Before the closure of the wound, the spring system must be aligned to where the first activation is to take place. The mucoperiosteal flap is placed over the traction construction and fixed in place with sutures. The end of the tension spring is temporarily fixed to the anchoring arch with suture material without tension.

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